Size: L130×W60×H135mm

Pantone Reflex Blue C

White

Black

CRDLIGHT

CE 0370 | FFP2 NR  
EN 149:2001+A1:2009

CRDLIGHT

25  
PCS

Filtrationseffizienz  $\geq 94\%$   
Filtration Efficiency

FILTERING HALF MASK  
FILTERNDE HALBMASKE

- Instructions:**
1. Press the mask against the face with the metallic strip uppermost
  2. Place the bands around both ears
  3. Mold the metal clip over nose bridge and the mask should snugly over the face
  4. Press the mask against the face with both hands to seal air leaks

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Product Features:

1. Efficient filtration: Filtering rate above 94% with pollen, droplet, dust, and smog effectively
2. 5 layers purification: Filament non-woven fabric, flexible moderate thickness, adsorption particle, double layer melt-blown filters
3. Smooth breathing: Flexible insert nosebar ensures optimal sealing between the bridge of the nose and the mask
4. Ergonomic design: Comfortable to wear, low impedance filter material and no odor
5. Secure seal: Soft metal nosebar for best seal between nasal beam and mask

Warnings:

1. Check the expiry date and mask before first use
2. Do not use after expiry date or the mask damaged
3. Do not use a microwave oven to heat it
4. Check and ensure the mask and straps were fit properly and tightly
5. The mask can not use in the space with poor ventilation due to the oxygen deficiency (below 19.5% vol)
6. Stop use mask when breathing is very difficult and change a new one
7. In order to keep the mask clean, avoid touching the inside of the mask with your hands
8. Not recommended for children under three years old due to low lung capacity
9. Please store the mask in a temperature range of -20°C to 25°C and a relative humidity of less than 80%

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Manufacturer: Jiangmen Yanyang Trading Co., Ltd.
Importer: Miko GmbH